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Manufacturer:

P.L. Superior Dental Materials GmbH, 22607 Hamburg, Germany

Instructions for Use**Calciumhydroxid-Liner****Low viscosity liner for pulp capping and underfillings**

Calciumhydroxid-Liner is a radiopaque water-based calcium hydroxide liner.

Indications

- direct capping when the pulp is opened or a pulpotomy is performed
- indirect capping for the treatment of caries profunda
- temporary filling of root canals
- lining of cavities to prevent exposure to acid media when using cements

Application

Clean and dry the cavity. If desired, the application needle might be bend in a slight arc to facilitate the application.

Handling precaution

Squeeze out the first small drop on a pad to ensure proper handling and prevent "shooting" out of the Calciumhydroxid-Liner by entrapped air. This must be done for each new application.

Apply Calciumhydroxid-Liner with a suitable dental instrument in thin layers until the desired thickness is reached. Allow each layer to dry briefly or dry carefully with an oil and water free stream of air. Do not apply to the cavity edges and remove any paste sticking to them. Then proceed with adhesive, underfilling and filling procedures.

For root canal fillings, first prepare and clean the root canal (e.g. hydrogen peroxide 3% or sodium hypochlorite solution) and insert Calciumhydroxid-Liner paste with a suitable instrument. Avoid overfilling (apical) and trapping of air. It is advisable to inspect the root canal filling by taking an X-ray.

Important Notes

Immediately after use stick the application needle of the syringe through the septum into the Anti-Dry Protector. The chemical and physical properties of the Anti-Dry Protector prevent hardening of Calciumhydroxid-Liner in the application tip and the Anti-Dry Protector acts also as disinfecting agent.

But for the next application it has to be used a **new application needle**.

Do not use hardened paste!

Side-effects

If the root canal is overfilled, discomfort may result (sensitivity for pressure and/or swelling) but, as a rule, this will subside within 24-48 hours.

Contra-indications or interactions are not known.

Warnings

- When using Calciumhydroxid-Liner wear protective glasses and cover the patient eyes.
- Calciumhydroxid-Liner is irritant. In case of contact with eyes a risk of serious damage to eyes exists.
- Avoid contact with skin, mucous membrane and eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- If accidental skin contact occurs, wash immediately with water.

Storage

Do not store at less than +8° C!

Do not use after expiry date.

After application stick the needle into the anti-dry protector.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!

Vertrieb:

Dr. Ihde Dental GmbH
Erfurter Strasse 19
D-85386 Eching
Tel. +49 89 319 761-0
info@ihde-dental.de

Manufacturer:

P.L. Superior Dental Materials GmbH, 22607 Hamburg, Germany