Deutsch:

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9999-D0039-03 V005

English:

Instruction for use – Please read carefully

IHDENT® Dentatrend® orange disinfection rapid

Type of preparation

According MPG: Ready aqueous surface disinfectant for the medical and dental field for rapid disinfection of non-invasive medical devices. Particularly suitable for use on alcohol-sensitive surfaces such as acrylic glass.

Application

The spray bottle is fitted with a dual-foam sprayhead. The surface disinfectant is applied as a foam (make sure that the outlet is located upstream of the valve of the spray head). The reconstituted solution is sprayed from a distance of about 20-30 cm as a foam on the surface to be disinfected, and distributed with a lint-free, undyed cloth evenly and allowed to act. Ensure complete wetting. Excess amount of preparation should be removed with an undyed cloth. Test the agent on sensitive materials such as artificial leather on a small, non visible area for compatibility. To avoid streaking, the treated surfaces should be periodically wiped with a damp (water), disposable cloth.

Microbiological effectiveness

IHDENT® Dentatrend® orange disinfection rapid effective against:

Bactericidal (incl. MRSA) according to VAH/DGHM, EN 1040, EN 1276, Levurocidal (Candida albicans) according to VAH/DGHM, EN 1650, EN 1275, EN 13697; "limited virucidal" (effective against enveloped viruses, i.e. HBV, HIV, HCV) and unenveloped Rota viruses and Noro viruses.

Contact time

Bacteria (incl. MRSA) acc. to VAH/DGHM, EN 1040, EN 1276, EN 13697:

high bio load conc. 1 min.

Candida albicans acc. to VAH/DGHM, EN 1275, EN 1650, EN 13697:

high bio load conc. 1 min.

Enveloped viruses (z.B. HBV, HIV, HCV) acc. to BGA(RKI)/DVV:

with bio load conc. 1 min.

Rota viruses acc. to BGVV/DVV:

conc. 1 min.

Noro viruses acc. to EN 14476: **low bio load**

conc. 30 min.

Product composition

100 g contains:

0,24 g Dialkyldimethylammoniumchloride

0,48 g Alkyl(Aryl)dimethylammoniumchloride

Keep out of reach of children.

Storage

Store the products at room temperature. Do not use after expiry date.

Instruction for disposal

Packing material should be completely emptied of contents before disposal.

Follow EWC 07 06 99 when disposing of any remaining product content.

Special note

Always read the Instructions for Use and the Material Safety Data Sheet before use.

Delivery units

5 L REF D0122

Expert opinions: Available on request.

Legend
 Read instructions

 Expiration date

 Charge number

CE 0029

9999-D0039-03

Revision issued 2015-07

IHDENT® and Dentatrend® are registered trademarks.

 **Dr. Ihde Dental AG**
Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by Dr. Ihde Dental GmbH
Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de

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