Deutsch:

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English:

Instructions for use – Please read carefully

## IHDENT® Etching Gel

IHDENT® Etching Gel is a very thixotropic etching gel which contains 37% phosphoric acid.

Deliverable in syringes of 2 ml each with very thin application needles, it can be very easily and precisely applied to the location that is intended to be etched.

### Constituents

o-phosphoric acid in water (37%), aerosil

### Therapeutic indications

Etching gel for the adhesive dental filling therapy and for etching the tooth surface for obtaining microretentions.

### Use

#### Preparation of the IHDENT® Etching Gel syringe

Unscrew the cap and replace with an application needle.

**Note:** In order to ease the application, the application needle can be bent slightly if necessary.

**Safety measures for use:** Press the first small drop on a mixing block to assure secure handling in order to avoid a possible squirting of the etching gel due to possible air bubbles.

Apply IHDENT® Etching Gel on the enamel edges of the cavity (for handling of the microtip, see next paragraph) and allow to act for 20 - 60 seconds. When using the total etch technique, begin the etching process on the enamel and then extend to the dentin so that the acting time on the enamel is between 20 and 60 seconds and the acting time on the dentin does not exceed 15 seconds.

Rinse at least 15 seconds with water. Afterwards dry with oil and water free air.

The dentin should not be dried too much since a moist dentin surface is important for the function of the adhesive system to be applied in following. The etched enamel margin must have a chalk white appearance. If this is not the case, the etching process must be repeated.

### Precautionary measures during etching

It is important that surfaces already etched are not contaminated. Should a contamination occur, the etching must be repeated as described above. Avoid contact of the etching gel with the skin, mucous membrane and eyes. Should accidental contact occur, rinse thoroughly with water.

### Application of the adhesive

A suitable adhesive can now be applied to the etched, saliva-free enamel and dentin surfaces with a brush in accordance with the instructions of the producer.

### Placing of the restoration, curing and finishing

Please refer to the according instructions of the producer.

### Side effects

IHDENT® Etching Gel may not be used in patients which have an allergy to this material. If an allergic reaction occurs, the application is to be discontinued immediately and the patient must consult a doctor immediately.

### Contraindications / Interactions

Underfilling materials can be etched by the etching substance.

### Precautionary statements

**P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

**P280** Wear protective gloves / protective clothing / eye protection / face protection.

### Warning

The contact on the product with the gingiva or other living tissue is to be avoided. If necessary, rinse thoroughly with water.

### Other instructions

Keep out of reach of children. Only to be used by the dentist.

### Storage instructions

Do not store over 25 °C. Do not use after the expiration date.

### Package size

Syringe of 2 ml each **REF 5138**

Legends	
	Read instructions
	Expiration date
	LOT Charge number
	Keep in a dry place
	Store tightly closed keep
	Temperatures up to 25 °C
	Manufacturer

CE 0029

**Danger**

**H314**

Causes severe skin burns and eye damage.

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IHDENT® is a registered trademark.
**Dr. Ihde Dental AG**